D

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EN

Plastic Capillaries end-to-end and Plastic Capillaries neutral (without anticoagulants)

PRODUCT DESCRIPTION

Intended use:	in vitro diagnostic medical device for capillary blood sampling collection with or without anticoagulant, sample container for collection and dispensing of human blood samples for in vitro diagnostics
Filling volume:	See packaging
Anticoagulant:	See packaging
Storage conditions:	2 °C - 30 °C / store in a cool dry place / protect from sunlight

INSTRUCTIONS FOR USE AND RESTRICTIONS

- The plastic capillary should only be used by trained medical personnel.
- Adhere to the applicable hygiene regulations when handling biological sample material. It may contain dangerous
pathogens. Protective gloves, safety glasses and suitable protective clothing must be worn. Risk of infection!
- The analysis should be carried out immediately after sample collection. If the sample is stored in the plastic capillary
before measurement, sedimentation may occur and this may affect the measured values.
- This product is not suitable for invasive applications!

Please refer to the relevant literature.

The plastic capillary is a Single use product and not suitable for reuse and must be properly disposed of after use.

BLOOD SAMPLE COLLECTION

1. Decisions relating to the puncture site and collection (including skin disinfection) should be made by the physician
carrying out the treatment.
2. Stimulate the blood flow in the puncture site area, for example, using a heat pack.
3. Perforate the skin so that drops of blood form quickly. Discard the first drop of blood. **Completely** fill the plastic
capillary with blood from the middle of the drop of blood without the formation of air bubbles.
4. **The analysis should be carried out immediately following the blood collection.**

Use in accordance with the instruction for use of the respective diagnostic kit/machine.

DISPOSAL

Dispose of the plastic capillary in accordance with the applicable local or laboratory regulations
(biological contaminated - hazardous waste)

	This product fulfils the requirements of the German Medical Devices Law and European Directive 98/79/EC for in vitro diagnostic medical devices
	In vitro diagnostic medical device
	For single blood collection only. Do not reuse or refurbish
	Store at 2 °C - 30 °C
	Store in a cool dry place
	Keep cool. Protect from sunlight
	Manufacturer: SC – Sanguis Counting Kontrollblutherstellungs- und Vertriebs GmbH Im Löchelchen 11 – 51588 Nümbrecht – Germany www.sanguis-counting.de/ifu/pcfbg sales@sanguis-counting.de +49 2293 9129 0

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