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## EN A-K-FIX Blood Sampler for Blood Gas sterile

<b>PRODUCT DESCRIPTION</b>	
Blood Sampler for Blood Gas with luer cap	
Intended use:	In vitro diagnostic medical device for arterial or capillary blood sampling containing an anticoagulant, sample container for the collection and dispensing of human blood samples for subsequent pH/blood gas, oximetry, electrolyte and metabolite analyses
Filling volume:	~ 230 µl
Anticoagulant:	~100 I.U./ml balanced lithium heparin
Storage conditions:	2 °C - 30 °C / store in a dry place / protect from sunlight

<b>INSTRUCTIONS FOR USE AND RESTRICTIONS</b>
<ul style="list-style-type: none"><li>The A-K-FIX may only be used by trained medical personnel.</li> <li>Adhere to the legally applicable hygiene regulations when handling biological sample material. It may contain dangerous pathogens. Protective gloves, safety glasses and suitable protective clothing must be worn. Risk of infection!</li> <li>The analysis should be carried out <b>immediately</b> after collecting the sample. If the sample is stored in the plastic capillary before measurement, sedimentation may occur and this may affect the measured values. This is especially the case for samples with high haematocrit levels, extreme measurement values, elevated auto-metabolism, or increased temperature!</li> <li>This product is not suitable for invasive applications!</li></ul>
Please refer to the relevant literature.
The A-K-FIX is are a Single use product an not suitable for reuse and must be properly disposed of after use.

<b>BLOOD SAMPLE COLLECTION</b>
<ol style="list-style-type: none"><li>Open the sterile packaging using the tear strip and use the product immediately.</li> <li>Attach the single-use needle 0.60 x 25 mm (23Gx1“) (see accessories).</li> <li>Decisions relating to the puncture site and collection (including skin disinfection) should be made by the physician carrying out the treatment.</li> <li>Perform the puncture; then allow the plastic capillary to fill completely with blood.</li> <li>End the puncture and withdraw the needle .</li> <li>Attach the provided luer cap.</li> <li><b>Immediately aspirate the sample into the analysis device or system and unscrew the posterior orange AUTO-STOP cap.</b></li></ol>

<b>DISPOSAL</b>
Dispose of the A-K-FIX in accordance with the applicable local or laboratory regulations (biological contaminated - hazardous waste!)
<p><b>LITERATURE</b></p> <p>IFCC <span> </span> “Approved IFCC Recommendations on Whole Blood Sampling, Transport and Storage for Simultaneous Determination of pH, Blood Gases and Electrolytes” Eur J Clin Chem Clin Biochem. 1995 Apr;33(4):247-53.</p> <p>AARC <span> </span> Clinical Practice Guideline “Blood Gas Analysis and Hemoximetry: 2013”, published on July 30, 2013 as DOI: 10.4187/ respcare.02786</p> <p>AARC <span> </span> Clinical Practice Guideline “Capillary Blood Gas Sampling for Neonatal &amp; Pediatric Patients” - Respiratory Care (Respir Care 2001;46(5):506–513)</p> <p>AARC <span> </span> Clinical Practice Guideline “Sampling for Arterial Blood Gas Analysis” - Respiratory Care (Respir Care 1992;37:913-917)</p>

<b>You can find further information in the blood gas/ISE analysis device manual.</b>
--

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